



### 1. ORDER FORM

Please read the attached forms carefully and fill in legibly with a ball pen. All documents have to be signed by test persons and/or their legal representatives. Enclose photos of all test persons.

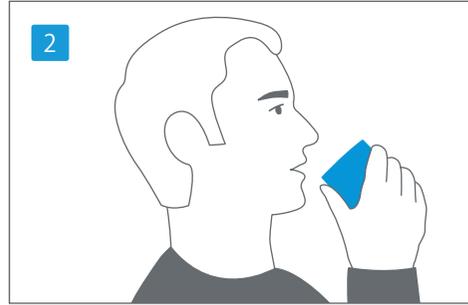


### 2. PREPARATION: RINSING

Mouth has to be clear of food debris – may be rinse with clear water. Please do not use any toothpaste. For breastfed children, please wait two hours after the last breastfeeding before collecting the sample.

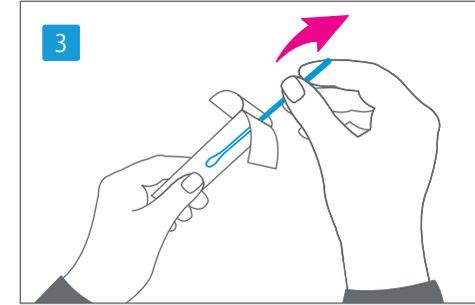


### 3. PREPARATION: COTTON SWABS

Get sample envelope ready for each person. Take the cotton swab out of the kit and carefully remove the protective foil. Each test person: 1 envelope, 2 cotton swabs.

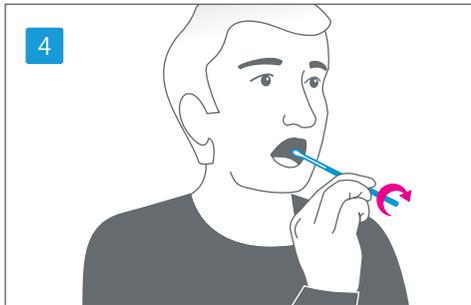


### 4. SAMPLE COLLECTION

Sample taking only in front of the witness. Use two cotton swabs per person. Touch only the end of the cotton swab and rub up and down the inside of the cheek of each person five times, with a rotating motion.

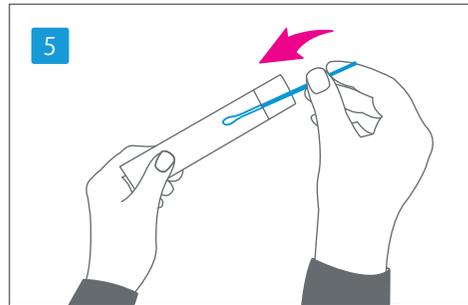

### 5. PACKAGING

Please insert both cotton swabs of a person (without protective foil) in the correspondingly marked sample envelope (do not mix up!) Place the envelopes as well as all completed documents in the box. Seal the parcel with the enclosed safety seal.

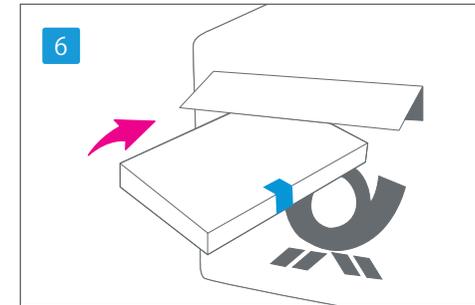

### 6. SHIPPING

Send the sealed test box to GENOLYTIC GmbH (only by witness (doctor, lawyer, pharmacy, authority)). Get the result online (5 days after payment and complete sample receipt). See login details on code card. Please write down the address on the back of form „proof of identity“ for the receipt of written certificate.

